

Scanning Strategy Tips

- Moving tissue is the issue: Retract the lips, cheeks and tongue for a better the experience.
- First path (**Occlusal**) is Key: spend a sec per tooth.
- To be exact, keep the scanner in contact.
- Constant and steady makes you happy.


Extra Tips:

- For Maxillary: rotate the tip so it's facing up
- When scanning the **Buccal** side: First start scanning with a **45°** angle then rescan with an **90°** angle.
- To properly fill the holes, angle the scanner in different direction.

Validation: 16/06/2021

3DISC Taking Digital Beyond

1. Occlusal 2. Buccal 3. Palatal

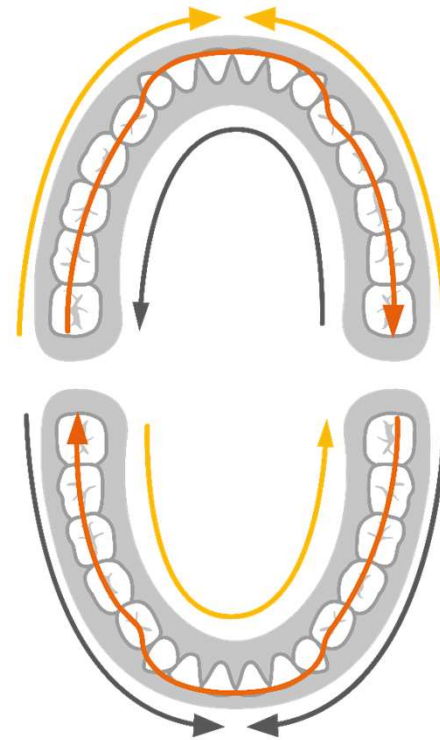

1. Occlusal 2. Lingual 3. Buccal .

1. Occlusal Line 2. Upper Gum
3. Lower Gum

