


Scanning Strategy Tips

- Moving tissue is the issue: Retract the lips, cheeks and tongue for a better the experience.
- First path (Occlusal) is Key: spend a sec per tooth.
- To be exact, keep the scanner in contact.
- Constant and steady makes you happy.


Extra Tips:

- For Maxillary: rotate the tip so it's facing up
- When scanning the **Buccal** side: First start scanning with a **45° angle** then rescan with an **90° angle**.
- To properly fill the holes, angle the scanner in different direction.


1. Occlusal 2. Buccal 3. Palatal

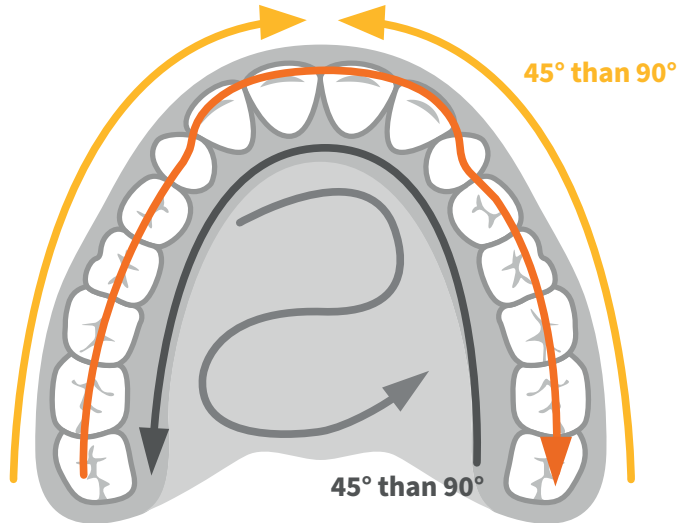

1. Occlusal 2. Lingual 3. Buccal

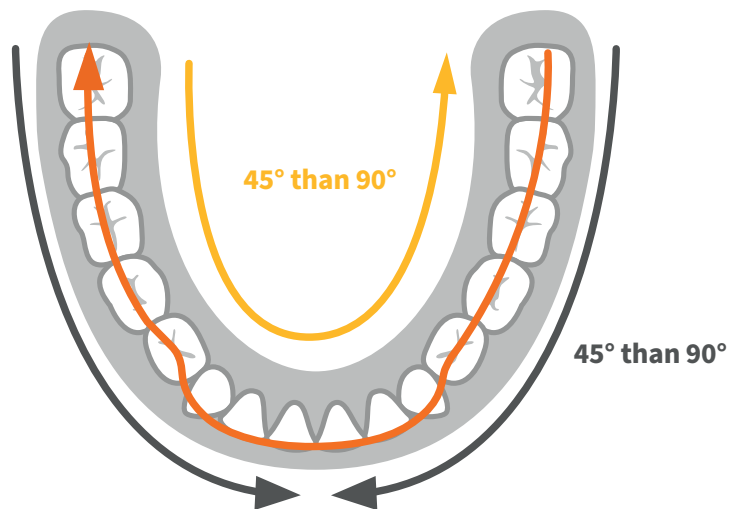

1. Occlusal line 2. Upper Gum 3. Lower Gum

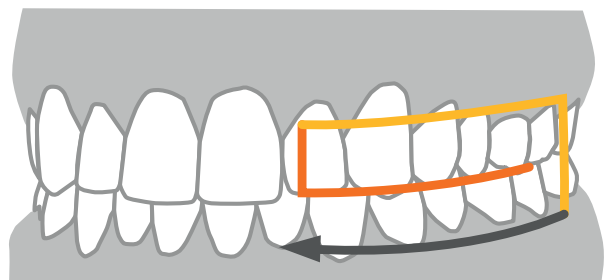

3DISC Taking Digital Beyond